VacSax BactiClear

EN VACSAX SUCTION BAG SYSTEM MANUAL

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
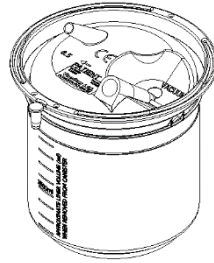
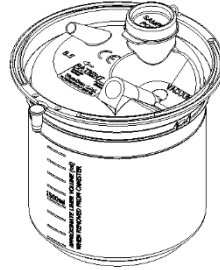
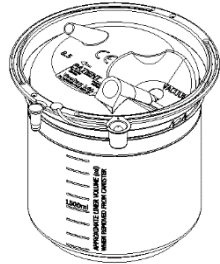
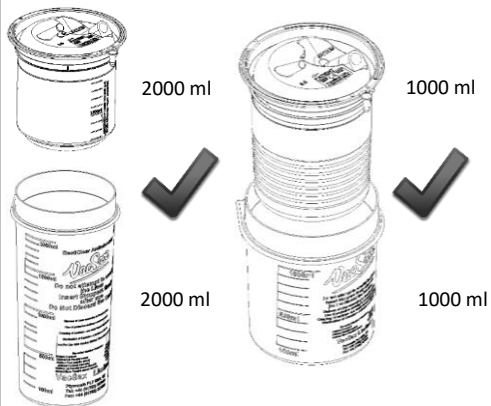
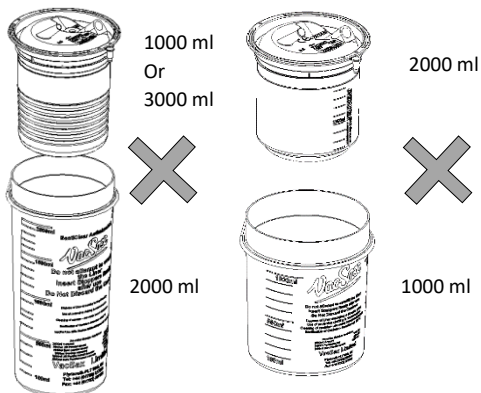
3

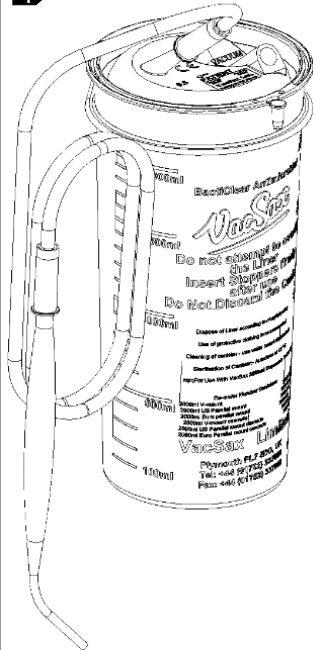4

CE Conforms to 93/42EEC Medical devices directive

Our commitment is to quality

VacSax Limited is an ISO 13485 registered company

Important information

Single use

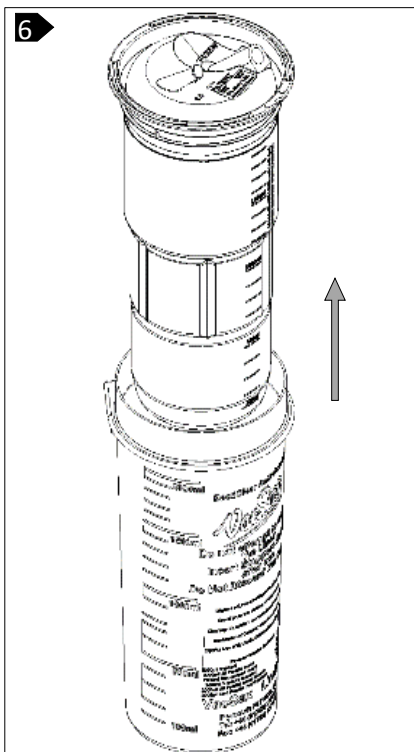
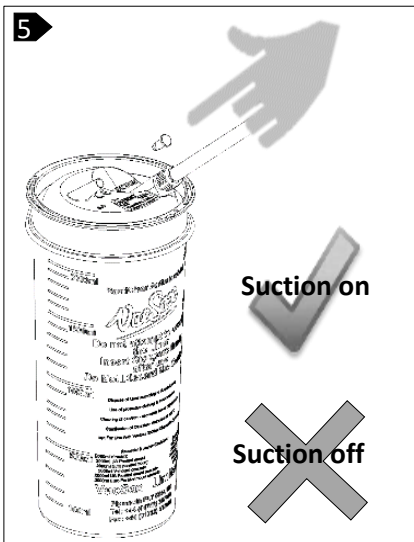
Packaging 100% Recycled 100% Recyclable

Latex Free

The BactiClear range of liners has been designed to aid the safe collection and disposal of waste fluids. The accessible component parts have been treated with a silver biocide tested to ISO 22196:2011

Intended use

The VacSax range of liners are intended for the safe collection and disposal of waste fluids and is only to be used by competently trained personnel.

Installation

Before use, ensure correct size of canister is available for the liner to be used. **1**

Ensure rigid canister is clean – wipe the inner rim surface with a damp cloth if necessary.

If using 3 litre liner, ensure correct tubing kit is used. (Pt No. 9910-353). When using sample port liner ensure large yellow port stopper is pushed firmly down.

Push liner firmly into canister **2**

Push taper connector into vacuum port with a twisting action **3**

Turn on suction and set to maximum to inflate bag. Reduce suction to desired level. If the liner bag fails to inflate or suction level is not achieved, replace liner.

Fit patient tubing and confirm vacuum is present at patient end **4**

After use

IMPORTANT! Do not turn off vacuum source while disconnecting patient tubing.

Remove patient tubing and fit stopper into port. **5**

Remove the vacuum connection by twisting the taper connector. **DO NOT PULL TUBING.** **▲**

Using the handles, lift the liner out of its canister. **6**

Dispose of vacuum tubing set if it becomes contaminated during use.

Pre use checks

- Check all liners are firmly pushed down into canisters.
- Check all tubing connectors are pushed firmly onto patient and vacuum ports.

Disposal

Users of VacSax products should be aware of the potential risk of contamination and must be familiar with national, local or hospital procedures for dealing with and disposal of potentially contaminated liquids and components.

Warnings

Check the product has not reached its use by date. This can be found printed on the liner and packaging label

Once a liner is installed check vacuum and flow regularly

The volume indication given by these devices is for reference only.

VacSax liners have been designed as single use, therefore they cannot be effectively decontaminated. The risk of cross contamination or infection may remain after reprocessing. The filter valve undergoes an irreversible conversion once wetted and cannot be reprocessed.

When using VacSax products with other equipment ensure all relevant instructions for use have been read

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VacSax Limited, Western Wood Way