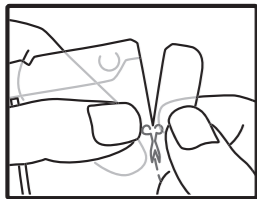
en**INSTRUCTIONS FOR USE****EASY STORE**
BREAST MILK STORAGE BAG

Package contents: 25 x Easy Store bags, Instructions for Use

Purpose:

The Easy Store breast milk storage bag is suitable for safe and compact storage and for easy heating of breast milk.

Important Information:

- The breast milk storage bag must be used once only.
- Thawed breast milk must not be refrozen.

Filling bag with breast milk:

1. Wash your hands thoroughly with soap and water and dry them before using the bag.
2. Label the green fields on the bag with a waterproof marking pen (name, date and time).
3. Cut the top end of the bag along the dotted line and open the zip closure.
4. Fill with breast milk to the 180 ml / 6 fl oz mark.
5. Press the air out and close the zip closure.

Storage of breast milk:

- At room temperature: 3 to 4 hours is optimal; 6 to 8 hours under very clean conditions at 16–29 °C (60–85 °F)
- In the refrigerator: 72 hours is optimal; 5 to 8 days under very clean conditions at approx. 4 °C (39 °F)
- In the freezer: 6 months is optimal; 12 months is acceptable at <–17 °C (0 °F)

Heating breast milk:

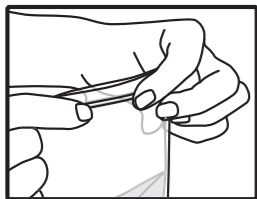
- The bag is not suitable for thawing and heating in a microwave oven.
- Thaw breast milk gently in a warm water bath or in the refrigerator.
- Heat the bag in a warm water bath and feed the breast milk as soon it reaches body temperature.

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