# Ardo natal support

**Ardo Natal Support Obstetric gel to facilitate vaginal birth**

Application only by trained personnel

**Indications and application:** Ardo Natal Support is a gel specially developed to facilitate vaginal birth. It is used for treatment during vaginal birth to facilitate vaginal birth for both mother and child and to protect the pelvic floor/perineum. Ardo Natal Support forms a lubricating film in the vaginal birth canal and thus reduces the birth-preventing frictional forces between the child and the vaginal birth canal. Ardo Natal Support is non-allergenic, electrically conductive, slightly acidic, isotonic, mucosa-compatible and supports sonographic examinations. The gel can be used for both natural births and vaginal operative births.

Ardo Natal Support facilitates and shortens the expulsion period and protects both the mother and the baby, especially the vagina, pelvic floor, perineum and the child.

In particular, Ardo Natal Support is used:

- To facilitate vaginal birth to protect both the mother and child by reducing the frictional forces that prevent birth in natural births such as water birth, premature birth
- For treatment of the protracted birth process
- For perineum protection during vaginal births
- For prevention of postpartum urogenital burning

**Special applications:** Use during the opening and expulsion period (stage I and stage II).

**Dosage and application:** Ardo Natal Support is applied during birth in stage I or stage II during a vaginal examination.

With the help of the flexible vaginal catheter or manually with the help of one hand, the full amount of a syringe is distributed in the vaginal birth canal, starting below the cervix. To achieve an optimal lubricating film, an additional NaCl solution can be added during gel application. Commonly used solutions such as NaCl or aseptic solutions can be used as fluids. Ardo Natal Support shortens the expulsion period. Application time is usually 30-45 minutes. The required dose is on average between 15 and 30 ml per birth. Experience has shown that the maximum dose of Ardo Natal Support is 30 ml.

**Type of application:** For local application in the vaginal birth canal.

**Precautions:** After opening the packaging, the vaginal catheter must be placed on the syringe. Preparation and application should be carried out under standard aseptic conditions.

After delivery of the infant's head, the eye-nose-facial region of the newborn should be wiped with a dry cloth. If necessary, superficial suctioning of the mouth/nose region should also be performed. Use a dry cloth during delivery to prevent the newborn from slipping.

Due to the electrical conductivity of Ardo Natal Support, electro-surgery can be used to stop bleeding at any time after use. Ardo Natal Support should therefore not be used with other gels to ensure the electrical conductivity of the gel.

**Biocompatibility and use in pregnancy:** There is no evidence of harmful effects on the mother or the newborn baby.

**Side effects:** There are no known side effects.

**Interactions:** There are no reported cases of interactions.

**Contraindications:** Ardo Natal Support should not be used in cases of:
- Suspected amniotic infection syndrome
- Signs of fetal asphyxia
- Known intolerance to any of the ingredients
- Contraindications for vaginal birth

**Storage and shelf life:** Ardo Natal Support should be stored at room temperature (+5 °C to +30 °C) and away from light. The expiry date is printed on the outer wrapping and on the package. Do not use Ardo Natal Support after the expiration date.

**Dosage form and content:** 1 disposable syringe containing 15ml obstetric gel individually packaged with vaginal catheter.

**Properties and galenic formulation:** Single-use packed and contains no medically active ingredients. Almost colorless gel of natural basis. The gel is highly viscous, isotonic with a slightly acidic pH value of 6.0 - 6.7, electrical conductivity 6.0 - 8.0 mS/cm.

**Special characteristics:**Ardo Natal Support supports the natural sliding effect during stage I (opening stage) and stage II (expulsion stage) of vaginal birth.

**Note to the user:** According to Regulation (EU) 2017/745 on medical devices, any serious incidents that occur in connection with the product must be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

**Also available:**

- Ardo Natal Perimassage
- Ardo Natal Restore
- Ardo Natal Anti-Stretch

Explanation of symbols:	
<span><span></span></span> <b>CE</b> mark	
<span><span></span></span> <b>MD</b> Medical device	
<span><span></span></span> Do not reuse	
<span><span></span></span> Observe the instructions for use	
<span><span></span></span> Batch number	
<span><span></span></span> Do not use if packaging is damaged	
<span><span></span></span> Temperature limit	
<span><span></span></span> Keep away from sunlight	
<span><span></span></span> Unique device identifier	
<span><span></span></span> Use by	
<span><span></span></span> Manufacturer	

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