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en **Intended use:** The kindestCup is a unique, multifunctional feeding cup for collecting and feeding breast milk. **Safety instructions:** The kindestCup consists of food-safe silicone and is safe for both mother and child. If you experience pain or feel unwell during use, please speak to a medical professional. Do not use the kindestCup as a replacement for breastfeeding.

The German text is the reference for the contents.

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en Collecting milk

Wash your hands and the kindestCup before use. Place the kindestCup against your breast and press it with your palm to squeeze out the air. You should hear the air escaping as you press. Make sure that the suction is sufficient to hold the kindestCup in place. To release the kindestCup, hold it firmly with one hand and push a finger between your skin and the rim of the cup. Attention: If the suction power is reduced or the baby knocks the kindestCup, it may come loose. The kindestCup is not intended to be worn over an extended period of time or as a bra pad.

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en Manually expressing milk

Wash your hands and the kindestCup before use. Make yourself comfortable. Hold the kindestCup in one hand and guide your nipple into the opening. Express the milk by massaging with the other hand. The kindestCup can be used in the left or right hand.

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en Feeding your baby

Hold your baby in a semi-upright position and support their head and neck. Move the kindestCup to the baby's bottom lip and allow your baby to feed on the breast milk on its own. Let your baby determine the speed and amount of each feed. Do not pour milk into your baby's mouth!

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