Compliant with EC 1935/2004 | Konform nach EC 1935/2004 | Conforme à la directive CE 1935/2004 | Conforme alla normativa CE 1935/2004 | Conforme a la Directiva CE 1935/2004 | Voldoet aan EC 1935/2004

Compliant with European directives | Konform nach europäischen Richtlinien | Conforme aux directives européennes | Conforme alle direttive europee | Conforme a las directivas europeas | Voldoet aan Europese richtlijnen

Operation / Betrieb / en fonctionnement / Esercizio / Funcionamiento / gebruik +5 °C (41 °F) - +40 °C (104 °F) | Transport / Transport/ transport / Trasporto / Transporte / transport -20 °C (-4 °F) - +50 °C (122 °F)

Relative humidity: 20–95% | Relative Luftfeuchte: 20–95 % | Humidité relative de l'air: 20–95 % | Umidità relativa: 20–95 % | Humedad relativa del aire: 20–95 % | Relatieve luchtvochtigheid: 20–95%

Keep packaging away from heat | Verpackung vor Hitze schützen | Mettre l'emballage à l'abri de la chaleur | Proteggere la confezione dal calore | Proteger el embalaje frente al calor | Bescherm de verpakking tegen hitte

Keep packaging away from moisture | Verpackung vor Nässe schützen | Mettre l'emballage à l'abri de l'humidité | Proteggere la confezione dall'umidità | Proteger el embalaje de la humedad | Bescherm de verpakking tegen vocht

Packaging: Caution, fragile | Verpackung: Vorsicht, zerbrechlich | Emballage: attention, fragile | Confezione: attenzione, fragile | Embalaje: precaución, frágil | Verpakking: voorzichtig, breekbaar

Ardo medical AG  
Gwerbestrasse 19  
6314 Unterägeri  
Switzerland

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en

The Easy Cup can be used by specialists and parents as an alternative feeding method to breastfeeding.

## Application

1. Sit down in an upright position, comfortably and relaxed.
2. Fill the Easy Cup if possible, at least up to half-full. Put a cloth under the baby's chin. Restless babies can be wrapped up in a towel.
3. It is imperative to hold the baby in an upright sitting position on your lap, so that you and the baby feel comfortable.
4. Place the lower rim of the Easy Cup gently against the lower lip of the baby. Do not exert any pressure on the lower lip.
5. Tip the beaker so that the milk touches the baby's lower lip. NEVER pour the milk directly into the mouth.
6. If the baby takes a break in drinking, leave the Easy Cup touching the lower lip.

## Manual cleaning

Before using for the first time and after each time it is used:

- Dismantle the product into its individual parts.
- Rinse for at least 1 minute using cold tap water.
- Clean the components of the product with commercial detergent and warm water (not over +40°C / 104 °F) and rinse with cold tap water. Make sure that all detergent residues are removed.
- A soft brush can also be used for heavy soiling.
- Allow all parts of a clean surface to air dry.

## Manual disinfection

1× daily after manual cleaning

- Make sure there is enough water in the pot in order to prevent damage/deformation.
- Boil the parts for 5 minutes.
- Allow the parts to cool down and dry on a clean surface.

Check the individual parts for cracks and changes to the surface according to the cleaning and disinfection process. If you discover any changes, dispose of the part affected and replace with a new one.

## Material

Beaker and lid consist of BPA-free polypropylene (PP)

## Disposal

Can be safely disposed of in compliance with local regulations.

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