## EN INSTRUCTION FOR INSTALLATION OF SUCTION LINER

### INTENDED USE

The Pergo Disposable Suction System is used for hygienic suction of various fluids and / or secretion from patients during medical treatment. The suction system has no contact with the human body. For single connections of Pergo-Suction Liner please follow the instructions according to Figures A1-A5.

For installation of serial connections please see Figures B1-B6.

Recommended application for solidifying agent (With Sample Port Models):

Dosage is dependent on the composition of the fluids. A high percentage of electrolytes in the solution typically requires a higher dosage. It is recommended to add extra solidifying agent if necessary.

### STORAGE AND TRANSPORTATION CONDITIONS

Products should be stored in a closed warm and dry environment. All products should be protected from damp, dirty and dusty environments. Products can be transported under ambient atmospheric conditions provided that they are protected from rain, dust and direct sunlight. Temperature-controlled transportation is not required.

### DISPOSAL

All disposable products used in the Pergo Disposable Suction System should be replaced for each patient. If long-term treatment is to be performed on a single patient, it is advisable to change the disposable products within 24 hours but at the latest after 72 hours. The products should be disposed in accordance with hospital disposal policy. **Turn off the vacuum after the suction procedure.**

### SHELF LIFE

Shelf life of Pergo-Suction Liner is 5 years. Shelf life of Pergo-Suction Liner with solidifying agent is 4 years.

### WARNINGS

The Pergo Disposable Suction System is only to be used by qualified healthcare professionals trained for the use of the suction system. Do not use any component of the system if there is any apparent damage. Pergo-Suction Liner must only be used with the same size of Pergo Canisters. Close all the ports before turning off the vacuum. Make sure connections are done correctly, according to Figure A or B series. Do not open the vertical fold.

Do not remove the suction liner before the vacuum reaches zero.

Please scan the QR Code for further details

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