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es

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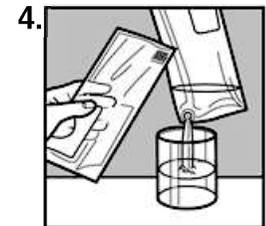
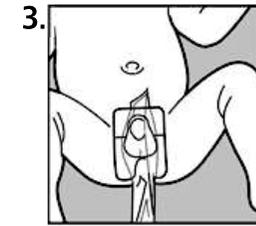
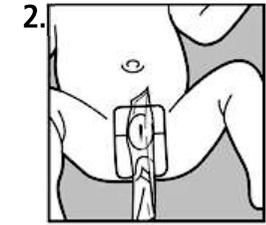
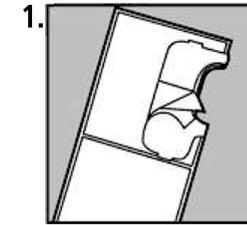
Unique Device Identifier

EC REP

Wellkang Ltd,  
Enterprise Hub, NW Business Complex,  
1 Beraghmore Road, Derry, BT48 8SE,  
Northern Ireland

CH REP

Ardo medical AG  
Gewerbstrasse 19  
6314 Unteraegeri  
Switzerland

Changzhou DSB Medical Co., Ltd  
#355, Longjin Road, Changzhou Economic  
Development Zone, Changzhou City,  
Jiangsu Province, China

0413

## INSTRUCTIONS FOR USE

### Urine Collection Bag

#### Applying the Single Specimen Urine Collection Bag

Separate the legs of the infant or child; make sure perineal area is clean and dry. Do not apply powders, oils or lotions to skin.

1. Remove the release paper from the lower half of the adhesive patch.
2. Stretch perineum to remove skin folds. Place the narrowest section of the exposed adhesive against the perineum. In female patients the perineum is the narrow bridge of skin between the vagina and the anus.
3. In males the perineum is the skin between the scrotum and the anus. Press the adhesive patch firmly against the skin to ensure proper adhesion. Working outward from this point, smooth the rest of the exposed adhesive against the skin. Avoid wrinkles which can create channels for leakage. Remove the release paper from the upper portion of the adhesive patch once the lower portion of the adhesive is securely in place. Working upward and outward, smooth the adhesive against the skin, pressing the adhesive firmly to avoid any wrinkles.

#### Removing and Draining the Urine Specimen Collection Bag

To remove the bag, start at the edge and gently peel the adhesive patch from the skin.

4. Tilt the bag so urine is away from the blue tab. Remove tab and drain into clean receptacle.

**Note:** Since the Preemie Urine Specimen Collection Bag has no drain tab, fold the adhesive patch in half so it adheres to itself and send the Urine Specimen Collection Bag to the lab.

**Caution:** As with any adhesive device, the adhesive component of these products may cause mild irritation upon removal.

**WARNING:** To help reduce the potential for infection and/or other complications, do not re-use.

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