CLINISTORE


Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso | Instrucciones de uso | Gebruiksaanwijzing

Ardo medical AG
Gewerbstrasse 19
6314 Unterägeri
Switzerland
T +41(0)41 754 70 70
F +41(0)41 754 70 71
info@ardo.ch
www.ardomedical.com

CE 0123 Class IIa

99.04.338 / V03 / 02.2021

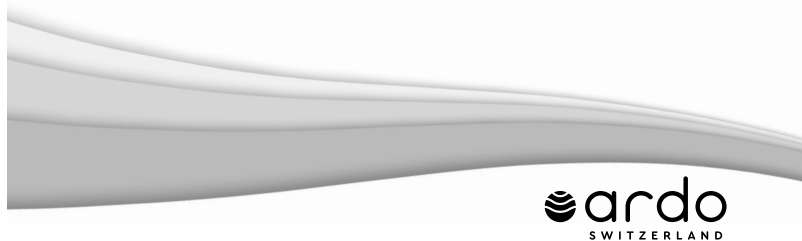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CLINISTORE Disposable milk bottles for hospitals

en

The disposable milk bottles are available in the sizes 50 ml, 80 ml, 120 ml and 240 ml. **Ready to use:** Ready-to-use bottles in bulk packaging.

Intended use

The disposable milk bottle is used for expressing, storing and feeding breast milk. It is intended for single use and should not be sterilized again or reused.

Expressing breast milk

Do not touch the inside of the disposable milk bottle and lid before use. To express breast milk, connect the disposable milk bottle to the original Ardo pumpset (see separate instructions for use).

Storage of breast milk

After expressing, close the disposable milk bottle using the lid. If necessary, the bottle can be labelled with the mother's name, the date, and the time of pumping. Breast milk can be stored as follows:

- at room temperature 16–29 °C (60–85 °F): 3–4 hours optimally; 6–8 hours are acceptable under very clean conditions
- in the refrigerator ≤ 4 °C (39 °F): 72 hours optimally; 5–8 days are acceptable under very clean conditions
- in the freezer < -17 °C (0 °F): 6 months optimally; 1 year is acceptable

Feeding breast milk

Thaw the breast milk which was stored in the disposable milk bottle in the freezer first in the refrigerator or at room temperature. Then warm the breast milk to about 37 °C (98 °F) by immersing in warm water or under lukewarm running water. Breast milk should not be heated in a microwave.

Materials

Disposable milk bottle: PP (polypropylene), lid: PE (polyethylene)
Disposal: According to the local regulations.

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