# GLASS BOTTLE

Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso | Instrucciones de uso | Gebruiksaanwijzing

99.03.712 / V04 / 07.2024

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For the collection and storage of breast milk.

en

## How to use these bottles

Expressing breast milk using a breast pump.  
Connect the bottle to an original Ardo Pumpset (see separate Instruction leaflet).

## Storage of breast milk

Seal the bottle with the lid provided. Breast milk can be stored as follows:  
At room temperature: up to 4 hours at 25°C (77°F); in the refrigerator: up to 4 days at 4°C (40°F); in the freezer compartment with door: 6 months up to 1 year at -18°C (0°F).

## Cleaning

Before using for the first time and after each time it is used:

- Dismantle the product into its individual parts
  - Rinse for at least 1 minute using cold tap water.
  - Clean the components of the bottle with commercial detergent and warm water (not over +40°C / 104 °F) and rinse with cold tap water. Make sure that all detergent residues are removed.
  - A soft brush can also be used for heavy soiling.
  - Allow all parts of a clean surface to air dry.
- Alternatively, you can also clean the product in the dishwasher.

## Disinfection

1x daily after manual cleaning

- Make sure there is enough water in the pot in order to prevent damage/deformation.
- Boil the parts for 5 minutes.
- Allow the parts to cool down and dry on a clean surface.

Alternatively, you can also disinfect the product in the microwave in a microwave bag (e.g. Ardo Easy Clean).

The product is also suitable for sterilisation with steam.

Check the individual parts for cracks and changes to the surface according to the cleaning and disinfection process. If you discover any changes, dispose of the part affected and replace with a new one.

## Material

Glass; Polypropylene

## Disposal

All parts must be disposed of in accordance with local regulations.

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