BREAST SHELL INSERT

Instructions for use | Gebrauchsanweisung | Mode d'emploi | Istruzioni per l'uso |
Instrucciones de uso | Instruções de utilização | Gebruiksaanwijzing | Bruksanvisning |
Käyttöohjeet | Bruksanvisning | Brugsanvisning | Инструкция по эксплуатации | Instrukcja
użycia | Οδηγίες χρήσης | Kullanım Kılavuzu | تعليمات الاستخدام | 사용 설명서 | 使用说明

	Languages		Page
en	Instructions for use	Breast shell insert	4-6
de	Gebrauchsanweisung	Brustglockeneinsatz	7-9
fr	Mode d'emploi	Embout de tétérelle	10-12
it	Istruzioni per l'uso	Inserto coppa	13-15
es	Instrucciones de uso	Adaptador de embudo para pecho	16-18
pt	Instruções de utilização	Adaptador para a copa	19-21
nl	Gebruiksaanwijzing	Borstschelp inzetstuk	22-24
sv	Bruksanvisning	Inlägg till brösttratt	25-27
fi	Käyttöohjeet	Maidonkerääjän lisäosa	28-30
no	Bruksanvisning	Brystskjoldinnlegg	31-33
da	Brugsanvisning	Indlæg til brysttragt	34-36
ru	Инструкция по эксплуатации	Прокладка для воронки	37-39
pl	Instrukcja użycia	Wkładka lejka	40-42
el	Οδηγίες χρήσης	Ένθετο χοάνης θήλαστρου	43-45
tr	Kullanım Kılavuzu	Meme huni eklentisi	46-48
ar	تعليمات الاستخدام	وليجة وعاء الثدي	49-51
ko	사용 설명서	갈때기 삽입물	52-54
zh	使用说明	内嵌式乳罩杯	55-57

BREAST SHELL INSERT

Sterile packed – Single Patient Use

1. Intended use

The breast shell insert is to be used in combination with Ardo pumpsets and is intended to be used by lactating women to express and collect milk from their breast. The breast shell insert is intended for single users only.


2. User risks

There are no contraindications known.

Safety regulations

The breast shell insert is to be used only in a sterilized or cleaned condition, and it must be checked for damage after each cleaning. The sterile breast shell insert must not be used if the packaging is damaged or opened prior to first use. If expressing is painful, seek professional help.

3. Assembling the breast shell insert

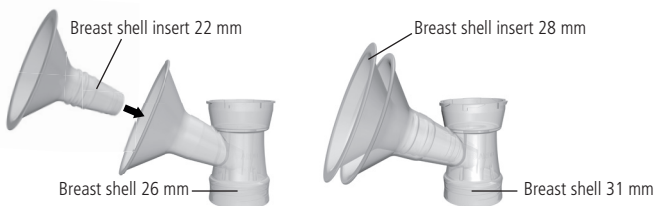4. Instructions for use

- The breast shell insert is intended to be used by only one person.
- Always wash your hands thoroughly with soap and water before handling.
- Position the breast shell insert in the breast shell with a gentle twisting movement.
- Select breast shell insert size 22 mm or 28 mm, to suit your nipple size. If your nipple rubs uncomfortably on the inner side of the funnel, use a larger breast shell / breast shell insert. If the areola is also sucked into the funnel, use a smaller breast shell / breast shell insert.
- The breast shell insert 22 mm fits breast shell 26 mm, and the breast shell insert 28 mm fits breast shell 31 mm.

5. Cleaning

The breast shell insert is delivered ready for use and can be used multiple times.

After every use

Wash the breast shell insert as soon as possible after expressing. Rinse it under cold running water and then either clean it by hand or in the dishwasher.

Clean by hand: Place the breast shell insert in a clean washing-up bowl used only for cleaning infant feeding items. Do not place it directly in the sink. Wash the breast shell insert with soapy hot water and a soft brush or clean dishcloth. After cleaning, rinse the breast shell insert well under running water.

Clean in a dishwasher: If possible run the dishwasher using hot water and a heated drying cycle. Remove it from the dishwasher with clean hands.

Air-dry: Allow the breast shell insert to air-dry in a clean area. Place it on an unused dish towel or paper towel. Do not use a dish towel to wipe or pat breast shell insert dry!

Storage: Store dry part in a clean, protected area until needed, e.g. sealable food storage bag.

Once a day

Sterilize the breast shell insert at least once daily after it has been cleaned. If you use a dishwasher with hot water and a heating drying cycle, separate sterilizing is not necessary.

Otherwise use one of the following options:

- dishwasher: with a sanitize setting or
- steam: use a microwave (only for a single user) or a plug-in steam system or
- boil: place the part into a pan and cover it with water. Boil for 5 to 10 minutes and remove item with clean tongs.

CAUTION: Let component cool after sterilizing before touching them.

6. General information

The breast shell insert is not suitable for autoclaving.

Materials: Polypropylene (PP) – can be safely disposed of in compliance with local regulations.

	Read the manual		Relative humidity: 20–93%
	Sterilized using ethylene oxide		Air pressure: 700–1060 hPa
	Temperature: Operation +5°C (41°F) to +40°C (104°F) / Transport –20°C (-4°F) to +50°C (122°F)		Use by date
	Do not use if package is damaged		Keep packaging away from heat
	Packaging: Caution, fragile		Keep packaging away from moisture

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Ardo medical AG